


ROOTT library integration in BlueSkyPlan software


Instructions that help you to import ROOTT library in BlueSkyPlan software



Step 1. Open BlueSkyPlan software, click **Tools - Manage catalogs** and download the latest catalog version.

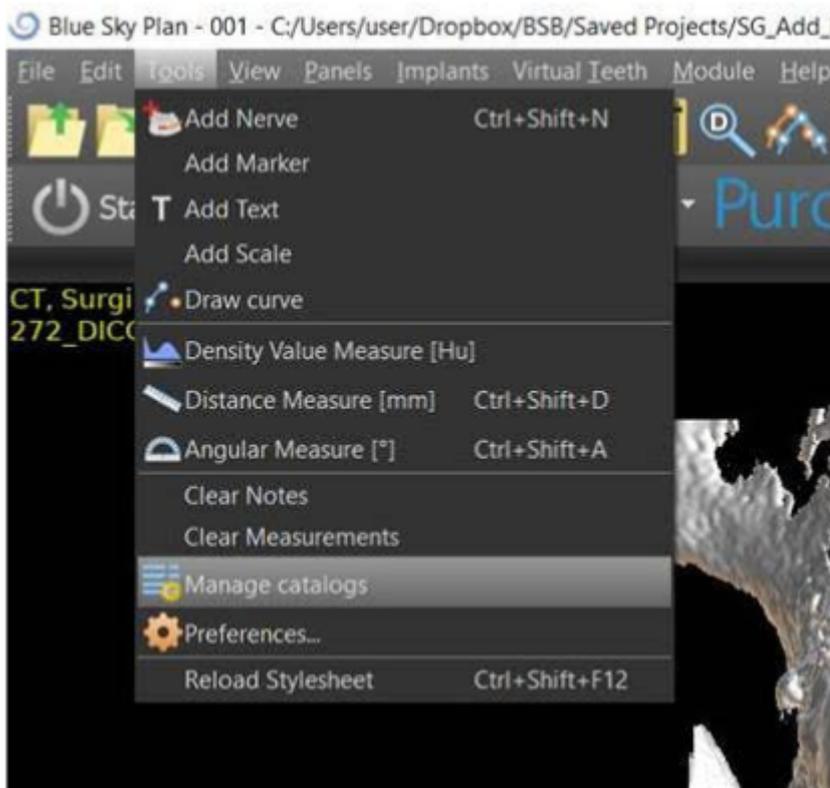


Step 2. Choose the downloaded file and click the check mark in the Section column.

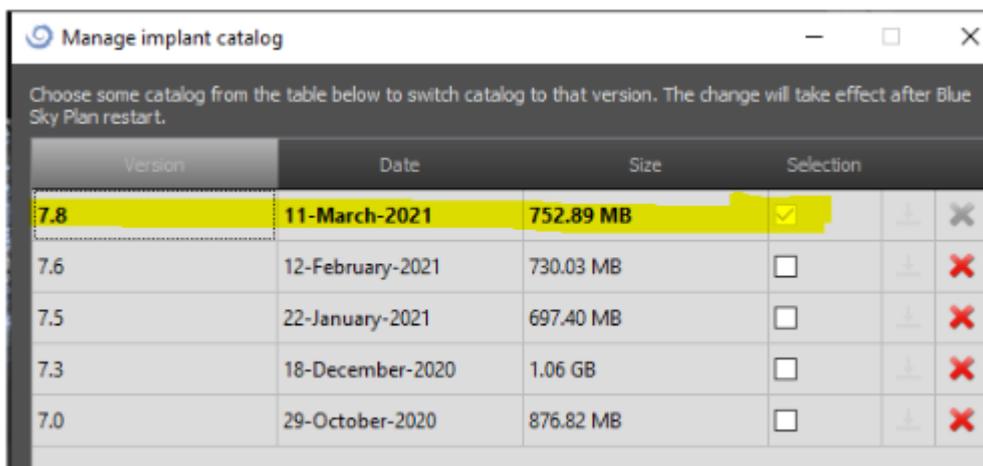


Step 3. Restart the system.

Optional. Use the BlueSkyPlan project that has a DICOM and loaded lower Jaw Model for testing: <https://bit.ly/3oGSNFR>.

Step 4. Uncover hidden parts by clicking **Add New Implant**.

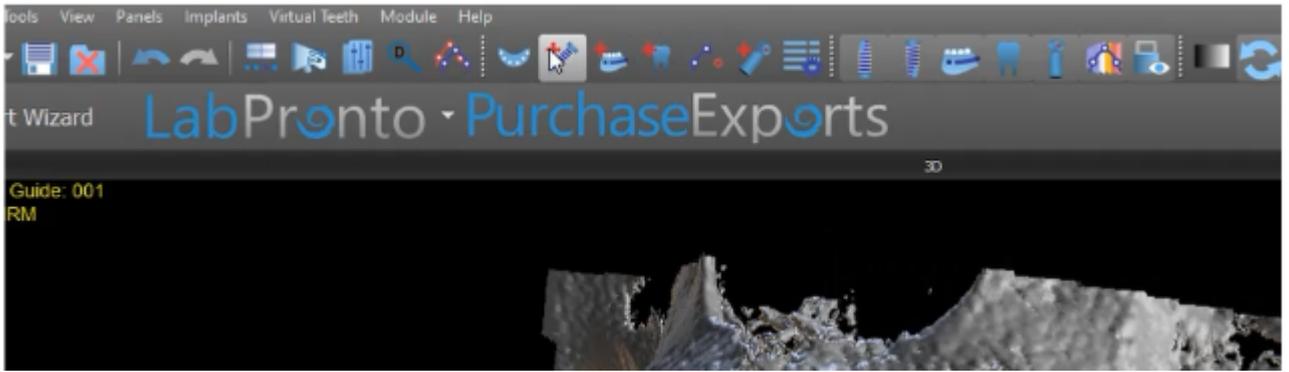

Step 5. Hold **Ctrl Shift** and click the Cancel button with the mouse. A new window shall appear.

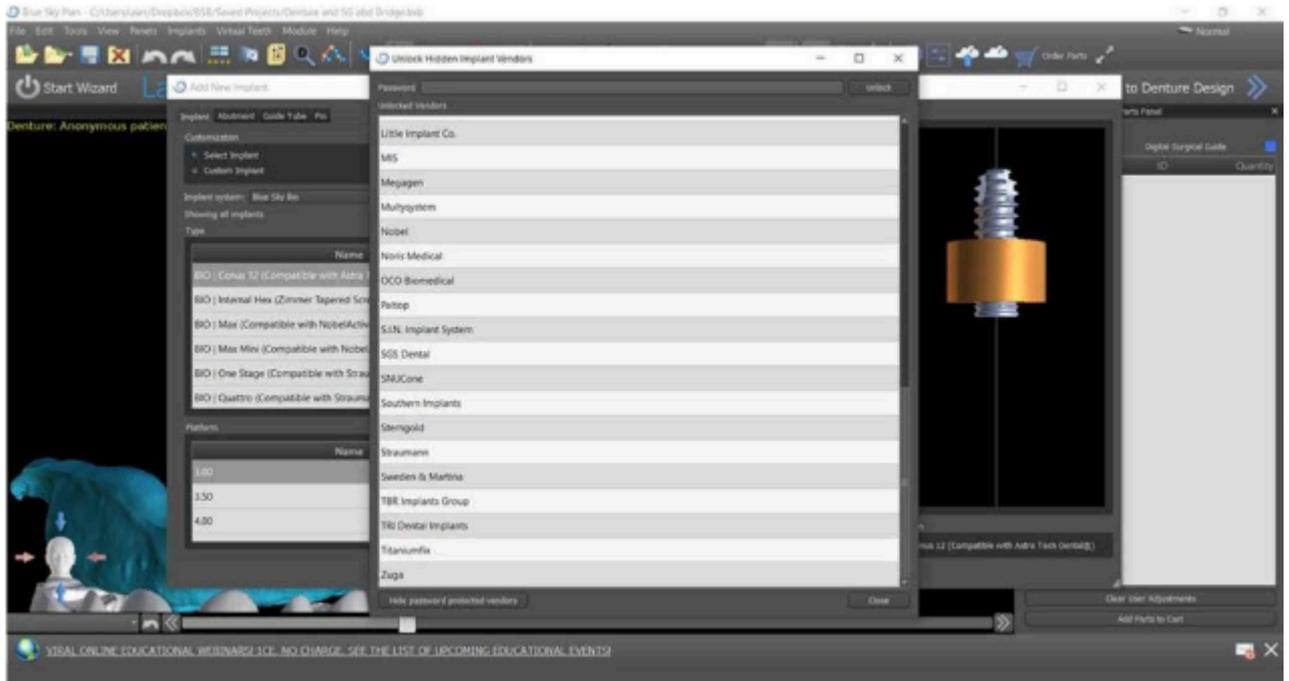

Step 6. Type **BSPTrate2021** inside the Password window and click **Unlock**.



ROOTT library now is visible in BlueSkyPlan software

For question please contact: msaltzman@blueskybio.com