 ROOTT implant library for  
Exocad software



ROOTT implant library –

installation instructions for


Exocad software

Instructions will help you to locate suitable ROOTT library folders with implants for  
Exocad libraries


Instructions are available in video format at  
[www.youtube.com](http://www.youtube.com)




**Step 1.** Find and open **EXOCAD** folder. Inside the folder locate and open **exocad- ChairsideCAD-64Bit**

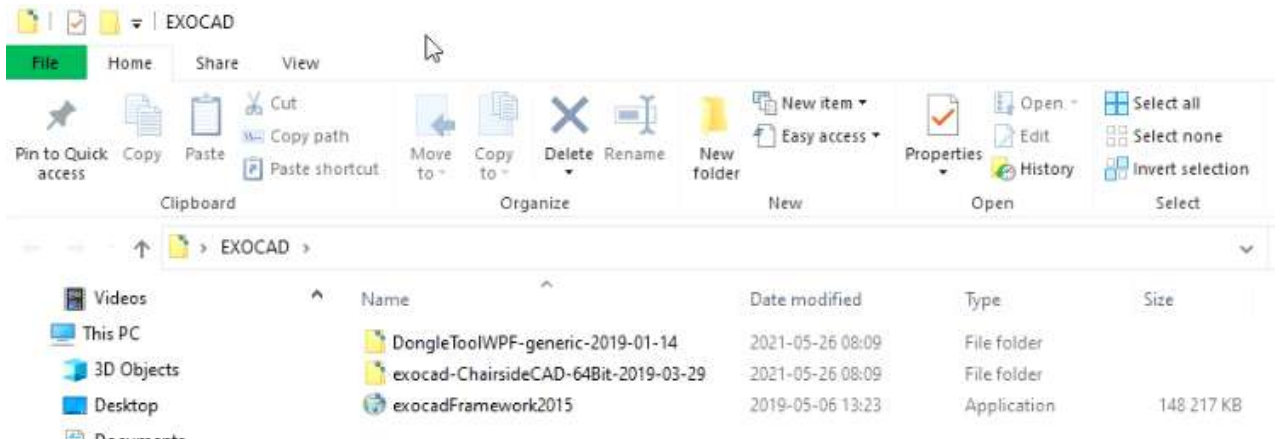

**Step 2.** Open **ChairsideCAD** folder

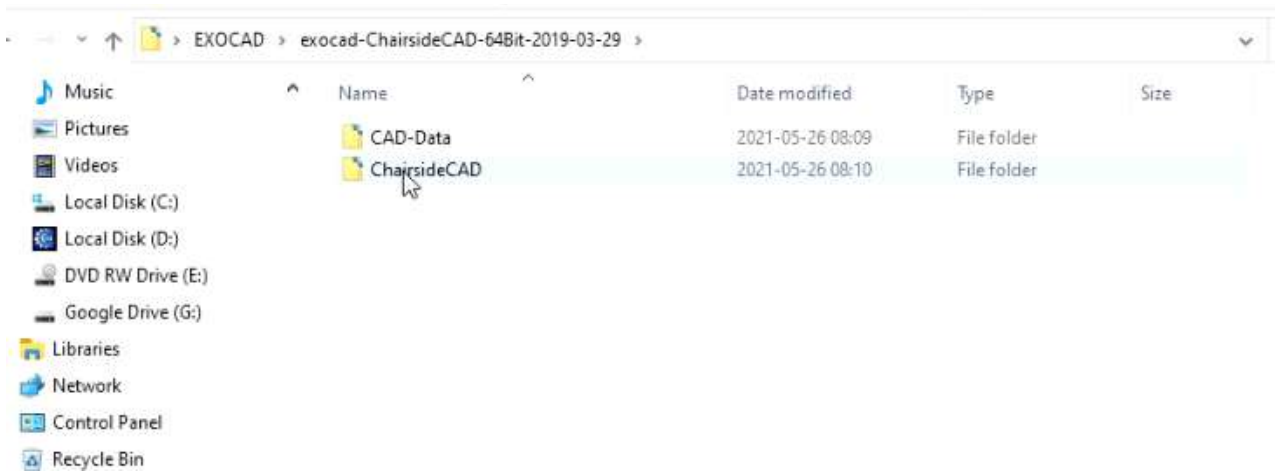


**Step 3.** Open **library** folder

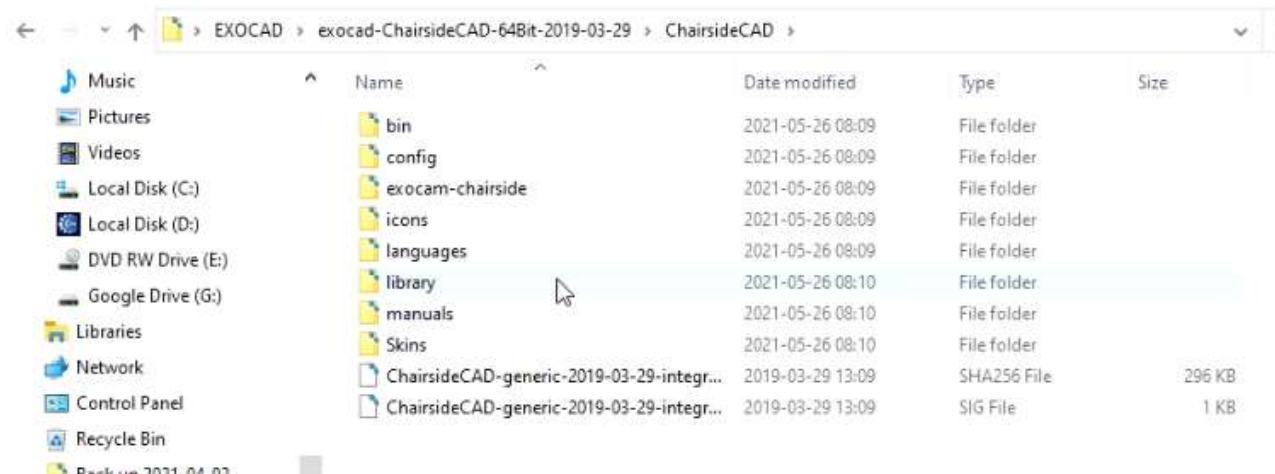


**Step 4.** Open **Downloads** folder (where ROOTT Library folder was downloaded) in new window




**Step 5.** Open **ROOTT Library** and open **Roott DentalCAD** folder

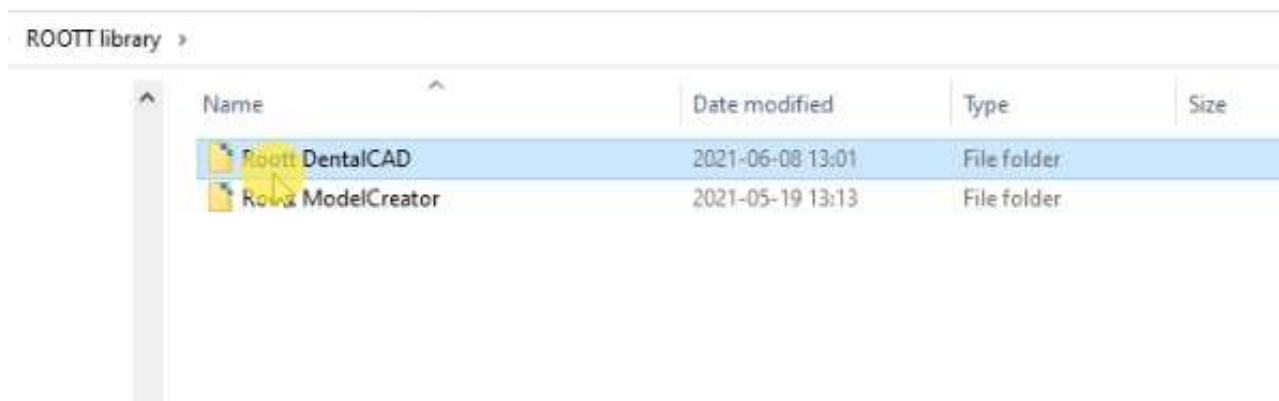


**Step 6.** Copy all files from **Roott DentalCAD** files

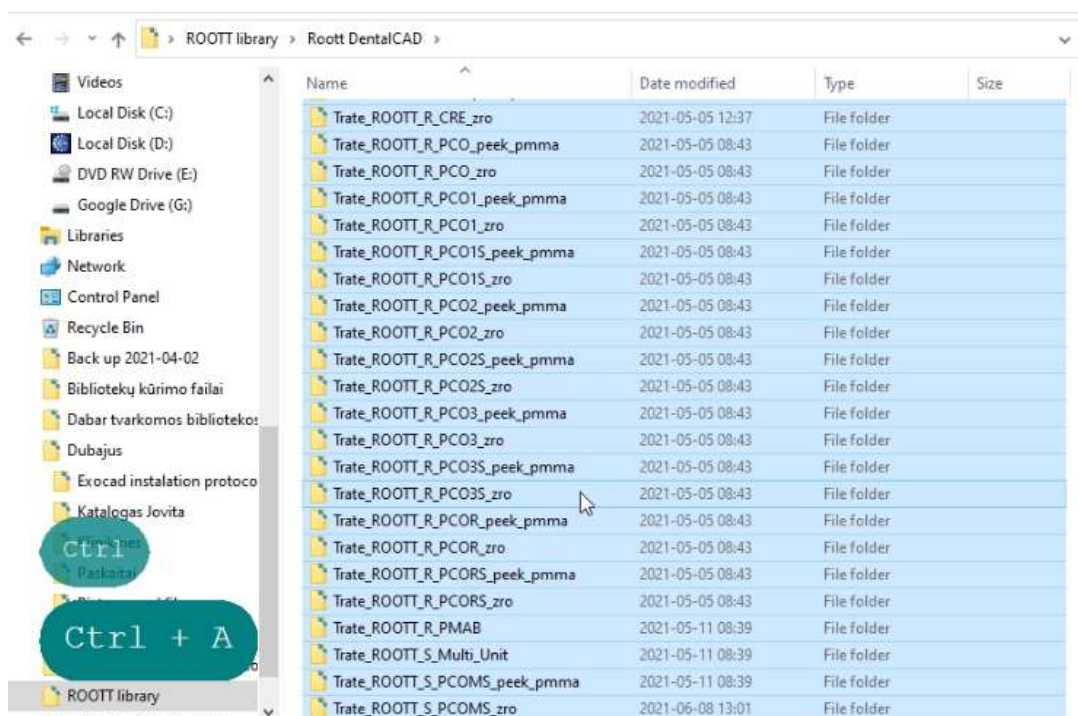


**Step 7.** Paste in **implant** folder (EXOCAD > exocad-ChairsideCAD-64Bit > ChairsideCAD > library > implant)

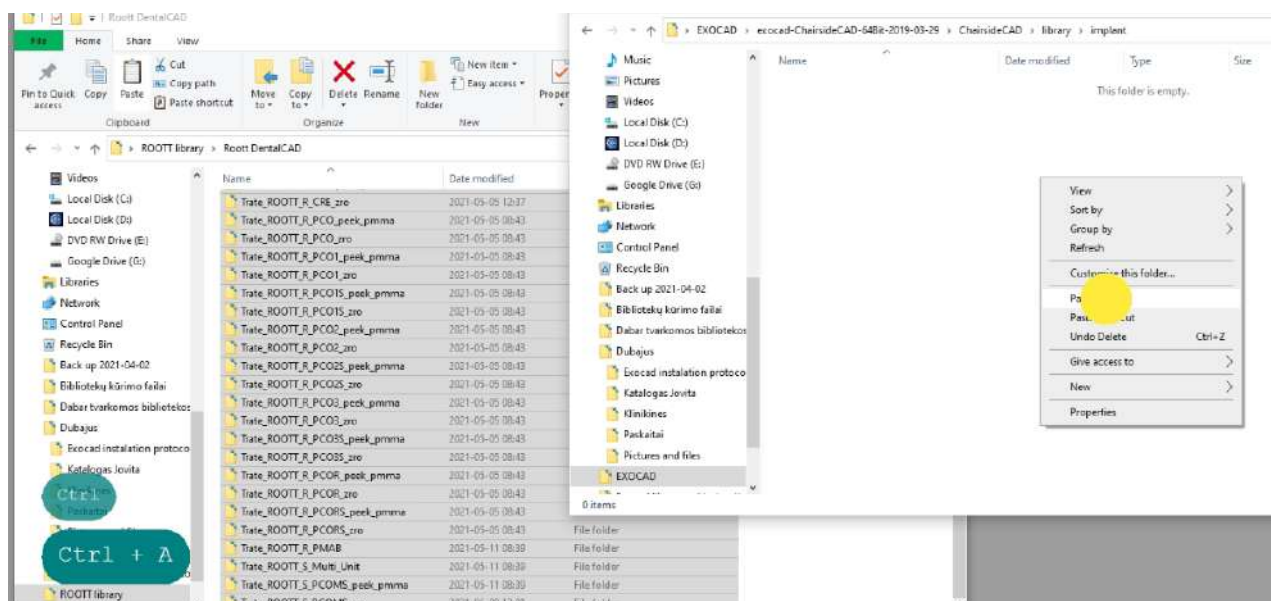


**Step 8.** Go back to **library** folder, choose **modelcreator** folder and then **implants** folder (implant < library > modelcreator > implants)

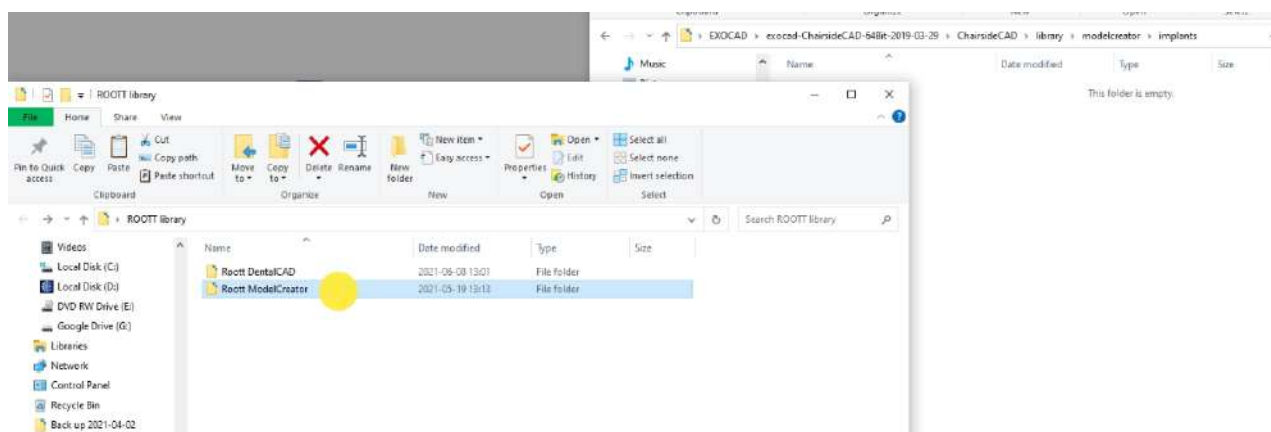

**Step 9.** In the second window go back to **ROOTT Library**, open **Roott ModelCreator** folder and copy 4 located folders (Roott DentalCAD > ROOTT Library > Roott ModelCreator)

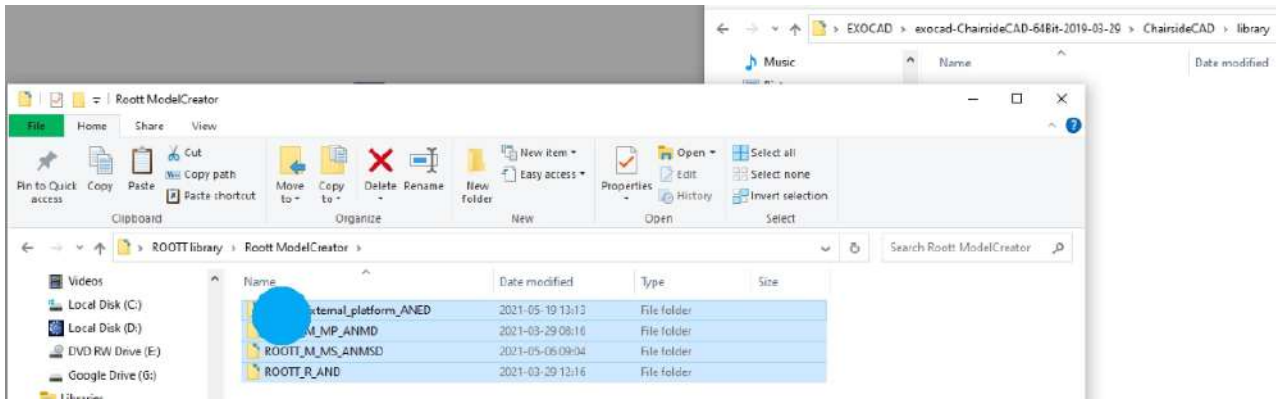

**Step 10.** Paste the copied folders in **implant** folder opened in the other window

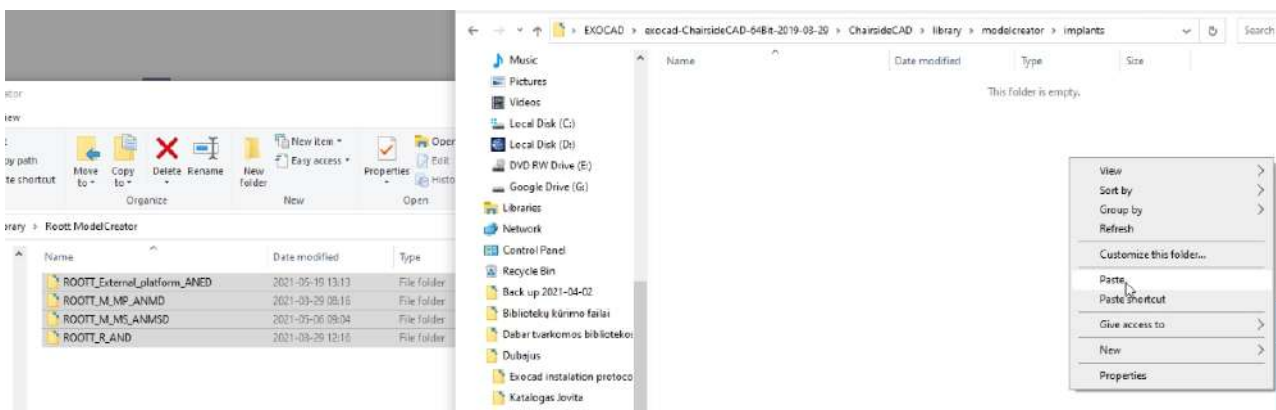

 ROOTT library is now available

For question please contact: [dalia.rimke@trate.com](mailto:dalia.rimke@trate.com)