

INSTRUCTION FOR OPENING BOXES AND BLISTERS OF ROOTT STERILE PRODUCTS



Make sure the package is not damaged before use.
Do not use if package is damaged.

The red color of the gamma-radiation sticker indicates that the package has passed the sterilization process. The same sticker protects the package from being opened from the bottom.



GAMMA RADIATION
STERILIZATION INDICATOR
RED IS γ EXPOSED

① Take a box in your hand



② Open the box: push & pull to open



③ Take the blister out



④ Take the blister in your hand



⑤ Open the blister



⑥ Take out the implant, holding only the green part

