

INSTRUCTION FOR OPENING BOXES AND BLISTERS OF ROOTT STERILE PRODUCTS



Make sure the package is not damaged before use.
Do not use if package is damaged.

- 1 Take a box in your hand



- 2 Open the box: push & pull to open



- 3 Take the blister out



- 4 Take the blister to the hand



- 5 Open the blister



- 6 Blister is opened, take the implant out.




Hold with fingers the green part as shown.
Pull up carefully holding on the green part.

- 7 Immediately insert clockwise the implant to prepared hole, holding only on the green part. Prepare hole before opening box.



- 8 Remove green holder by side to side slow motion and lift up.

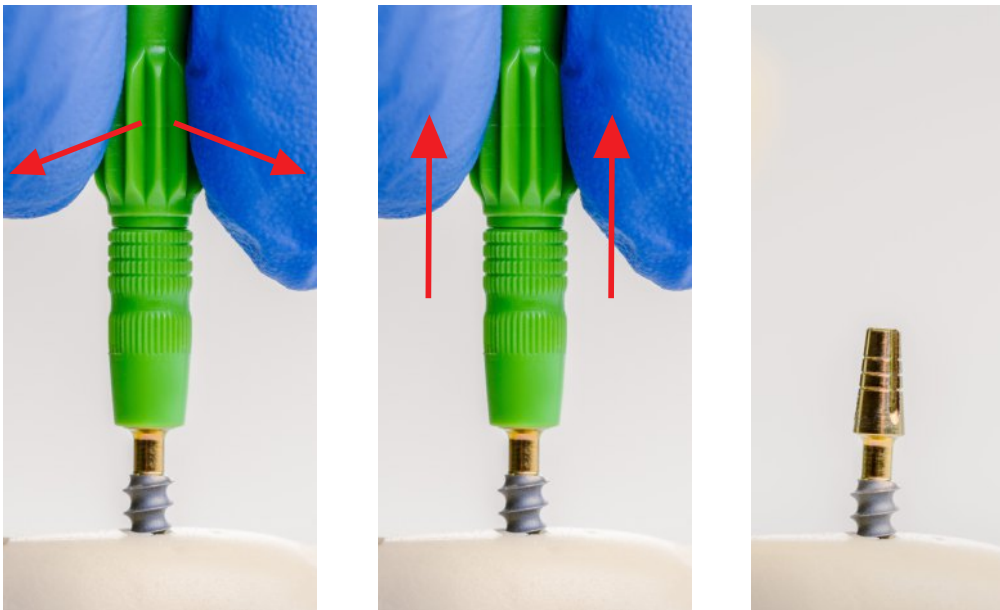

For next steps please read "*Placement protocol*".