

DW – handle for implant driver

instruction manual



Intended use


The device is used for supporting implant drivers that are used with a wrench.

Main features

- More precise implant insertion procedure.
- Less chance for the insertion tool to slip from an implant.
- No need to hold the wrench with a finger. Nothing obstructs the view, especially in distal areas.
- No chance to grab the glove while using the implant driver.

How to use

Put the instrument inside a hole in the ratchet instrument head and hold it with your finger where it is comfortable, then with the other hand, turn the wrench clockwise.

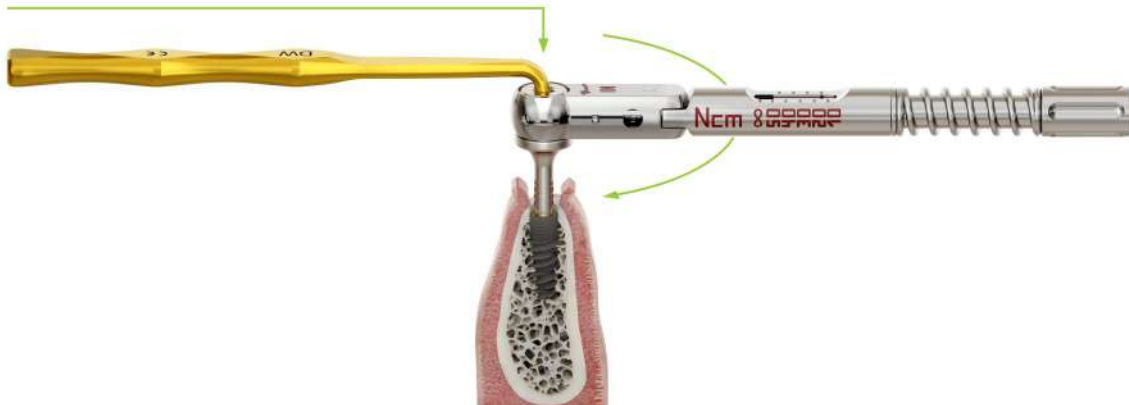

Can be used with these ratchet instruments

Implant drivers **R**



IT

ITL

Implant drivers **C CS B BS**



ITES

ITE

ITL

ITEXL

Implant drivers **K**



ITEKS

ITEK

ITEKL

ITEKXL



ROOTT

TRATE AG
Bahnhofstrasse 16
6037 Root
Switzerland

+41 41 450 01 01
info@trate.com
trate.com

TRATE

Medical devices under these instructions are in compliance with established in EU regulatory requirements.



TRATE UAB
Jonavos st. 254
44110 Kaunas
Lithuania



TRATE AG
Bahnhofstrasse 16
6037 Root
Switzerland