

# ROOTT S IMPLANT PLACEMENT PROTOCOL

## from box opening to complete implant insertion

1 Take a box in your hand



2 Open the box: push & pull to open



3 Take the blister out



4 Carefully open the blister



5 Take out the implant, holding only the green part





6 Immediately insert clockwise implant to prepared hole, just after opened the box, holding on green holder



Green holder is not intended to be used as an implant driver

7 Remove green holder by side to side slow motion and lift up

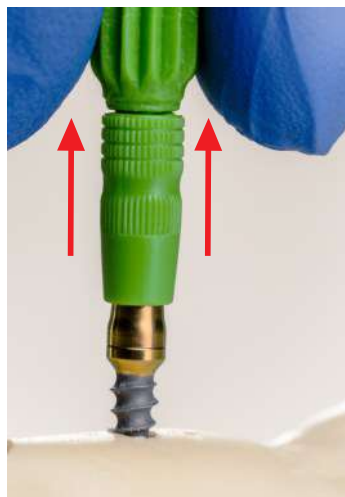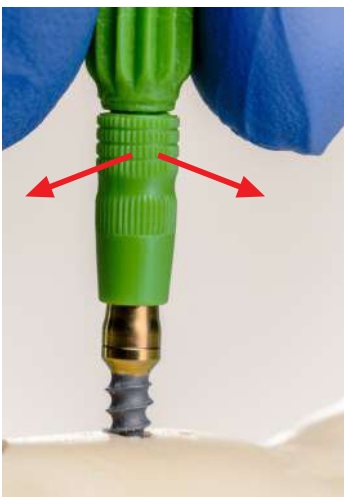

- 8 Insert implant via CREM with implant driver **C CS B BS** . Use implant driver ITES/ITE/ITEL/ITEXL for ratchet TW50 with torque up to 50 Ncm



Or implant driver ITEH/ITEHL for handpiece, use contra-angle handpiece



Or implant driver ITEAO for AO connection, use ETAO handle



Implant insertion via CREM carrier part - never exceed 93 Ncm


- 9 Remove CREM.  
Unscrew SCREM screw with screwdriver **U** and take off CREM.  
Use SD/SDL/SDXL for ratchet TW50



Or screwdriver SDH/  
SDHL/SDHXL for handpiece,  
use contra-angle handpiece



Or screwdriver SDAO  
for AO connection,  
use ETAO handle

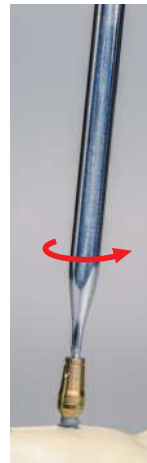

Remove the screw and implant carrier



- 10 Insert implant via direct insertion with implant driver **S**. Use implant driver ITMS/ITMSL/ITMSXL for ratchet TW50 with torque up to 50 Ncm.



Implant insertion via direct insertion never exceed 133 Ncm

- 11 Implant inserted in crestal position

