

ROOTT^R IMPLANT (with breaking point) PLACEMENT PROTOCOL

from box opening to complete implant insertion

1. Take a box in your hands. Push and pull to open



2. Take the blister out



3. Carefully open the blister



4. Take out the implant, holding only the green part



5. Take implant driver R and insert it to the implant. Make sure the tool is inserted firmly.

Use implant driver IT/ITL

or ITHS/ITH/ITHL




6. The holder can be twisted off or broken off. We recommend to twist off the holder.

a) Hold the instrument tight with one hand and with the other hold the holder, twisting until the implant separates from the holder.

Hold with implant driver IT/ITL

or ITHS/ITH/ITHL

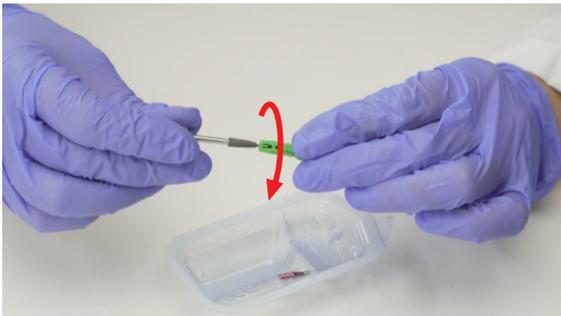

b) Hold the instrument tight with one hand and with the other hold the holder and break it off.

Hold with implant driver IT/ITL

or ITHS/ITH/ITHL



7. After twisting or breaking, the implant is separated from the holder.



8. Immediately insert implant directly to the prepared hole with implant driver R.


Use implant driver IT/ITL for ratchet TW50 or TW70 with torque up to 50-70 Ncm.

Or implant driver ITHS/ITH/ITHL for handpiece, use contra-angle handpiece.



Implant insertion via direct insertion - never exceed 100 Ncm.

9. Implant inserted in subcrestal position.

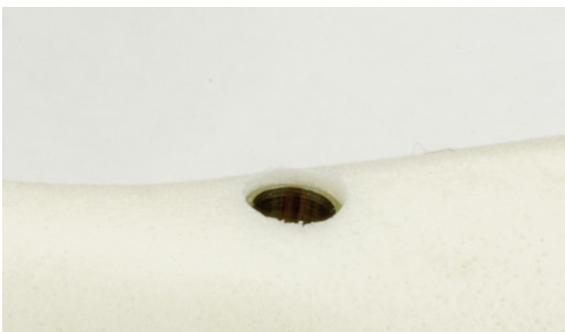

10. a) Implant can be covered with covering screw SC.

Unscrew a cover screw SC from the green holder. Manually unscrew with screwdriver SD/SDL/SDXL



Screw cover screw SC to cover the implant. With screwdriver SD/SDL/SDXL use ratchet TW50 or TW70, torque up to 15 Ncm




Implant covered with cover screw



b) or can be made other prosthetic actions depending on specific situation and choosing Related Dental superstructures

11. In the package can be found ZeroBoneLoss abutment ZBL with S8 screw.

Please note that screw S8 is screwed in opposite side of ZBL

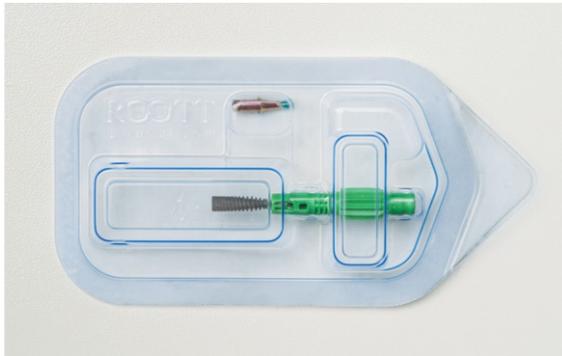

Unscrew S8 screw manually with any screwdriver U (SD/SDL/SDXL)

