

ROOTT M IMPLANT (with breaking point) PLACEMENT PROTOCOL

from box opening to complete implant insertion

1 Take a box in your hands



Push and pull to open




2 Take the blister out



3 Carefully open the blister



- 4 Take out the implant, holding only the green part

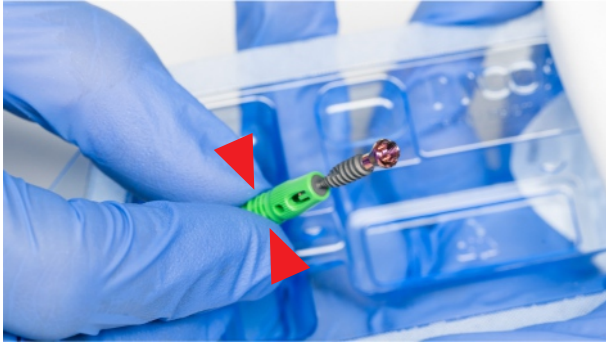



- 5 Take implant driver M/P and put on the implant. Fixate instrument with screw (screw is packed with implant driver) Make sure the tool is mounted firmly. Use implant driver ITM0/ITM10/ITM/ITML/ITMXL.



- 6 The holder can be twisted off or broken off. We recommend to twist off the holder.

a) Hold the instrument tight with one hand and with the other hold the holder, twisting until the implant separates from the holder.

b) Hold the instrument tight with one hand and with the other hold the holder and break it off.

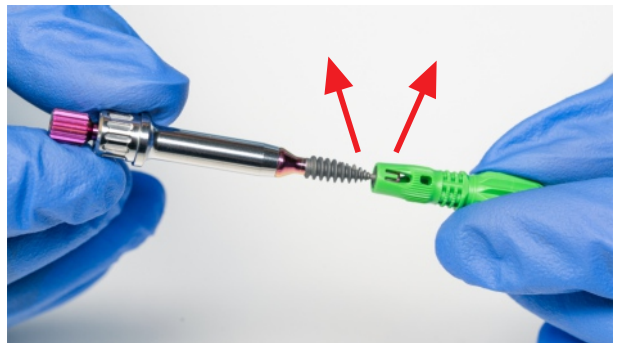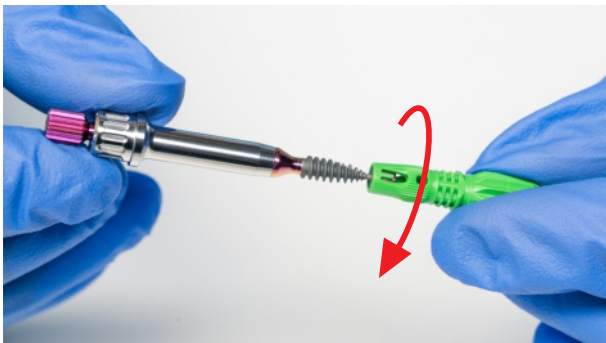

- 7 After twisting or breaking, the implant is separated from the holder.



- 8 Immediately Insert implant directly to the prepared hole with implant driver M/P. Use implant driver ITM0/ITM10/ITM/ITML/ITMXL for ratchet TW50 with torque up to 50 Ncm.



Implant insertion via direct insertion with insertion tools never exceed 133 Ncm.

- 9 Implant inserted in crestal position.

