

ROOTT B/BS IMPLANT PLACEMENT PROTOCOL

from box opening to complete implant insertion

① Take a box in your hand



② Open the box: push & pull to open




③ Take the blister out



④ Carefully open the blister



- 5 Take out the implant, holding only the green part


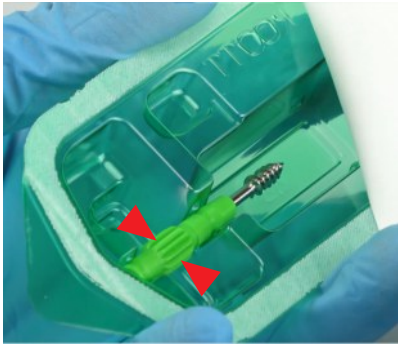


- 6 Immediately insert clockwise implant to prepared hole, just after opened the box, holding on green holder



Green holder is not intended to be used as an implant driver

- 7 Remove green holder by side to side slow motion and lift up

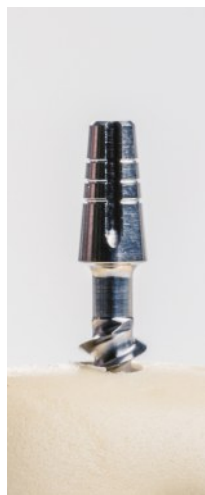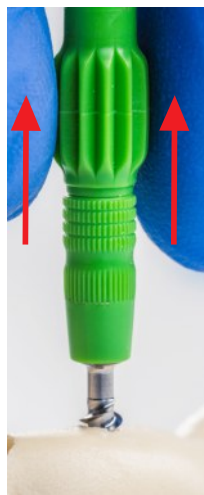

8 Insert implant with implant driver **C CS B BS** .


Use implant driver ITES/ITE/ITEL/ITEXL for ratchet TW50 with torque up to 50 Ncm



Or implant driver ITEH/ITEHL for handpiece, use contra-angle handpiece



Or implant driver ITEAO for AO connection, use ETAO handle

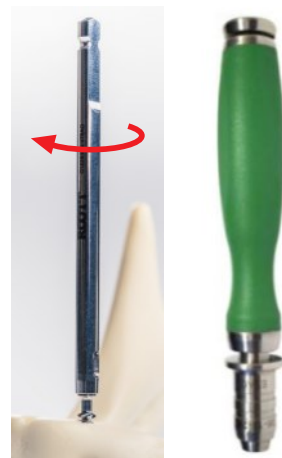

Implant insertion via direct insertion never exceed 117 Ncm

- 9 a) Implant inserted
in crestal position



- b) Implant inserted
in subcrestal position



- 10 Implant (except with 1.5 mm neck) can be bent up to 15 degrees with
abutment bender **C** **B** **BS** BT, push on top and bend (avoid vertical forces)

