

ROOTT C/CS IMPLANT PLACEMENT PROTOCOL from box opening to complete implant insertion


① Take a box in your hand



② Open the box: push & pull to open




③ Take the blister out

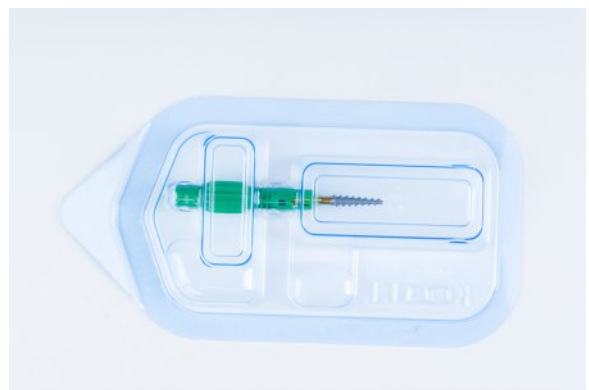

④ Carefully open the blister



- 5 Take out the implant, holding only the green part

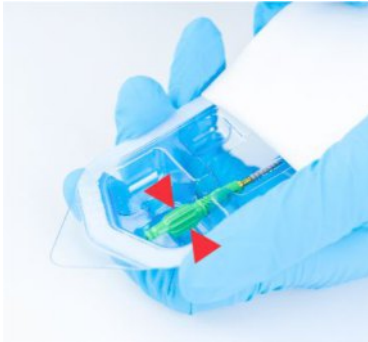


- 6 Immediately insert clockwise implant to prepared hole, just after opened the box, holding on green holder



Green holder is not intended to be used as an implant driver

- 7 Remove green holder by side to side slow motion and lift up

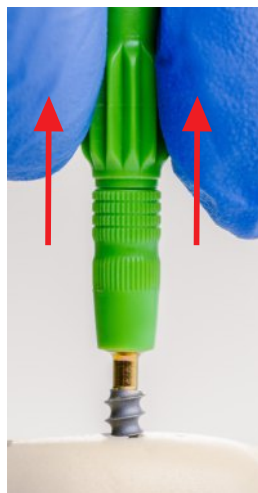

- 8 Insert implant with implant driver **C CS B BS** . Use implant driver ITES/ITE/ITEL/ITEXL for ratchet TW50 with torque up to 50 Ncm



Or implant driver ITEH/ITEHL for handpiece, use contra-angle handpiece




Or implant driver ITEAO for AO connection, use ETAO handle



Implant insertion via direct insertion:

- from diameter 3.0 mm never exceed 117 Ncm
- from diameter 3.5 mm never exceed 133 Ncm
- from diameter 4.5-5.0 mm never exceed 238 Ncm
- from diameter 5.5 mm never exceed 298 Ncm


- 9 a) Implant inserted
in crestal position

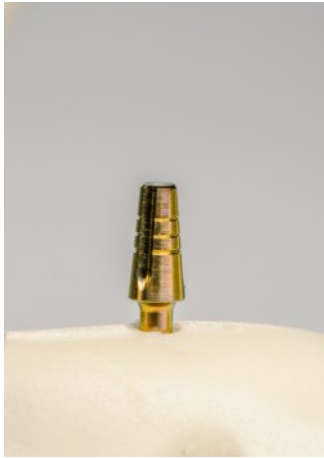

- b) Implant inserted
in subcrestal position



- 10 Implant ROOTT C can be bent up to 15 degrees with abutment
bender **C** **B** **BS** BT, push on top and bend (avoid vertical forces)

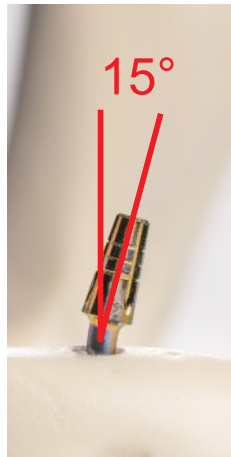

Only ROOTT C can be bent