

ROOTT M/P IMPLANT PLACEMENT PROTOCOL

from box opening to complete implant insertion

1 Take a box in your hand



2 Open the box: push & pull to open



3 Take the blister out




4 Carefully open the blister



5 Take out the implant, holding only the green part






6 Immediately insert clockwise implant to prepared hole, just after opened the box, holding on green holder

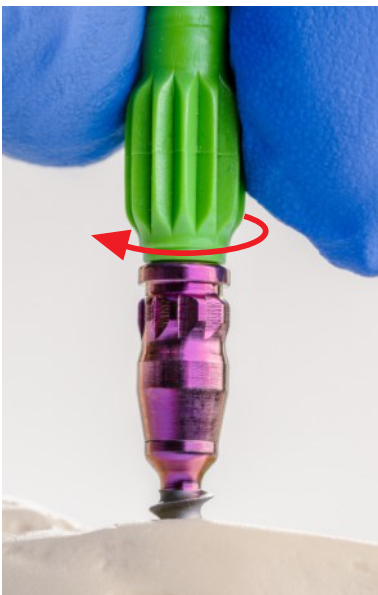

Green holder is not intended to be used as an implant driver

7 Remove green holder by side to side slow motion and lift up

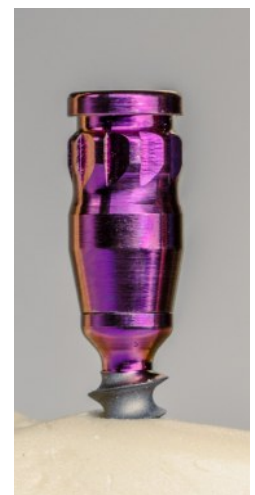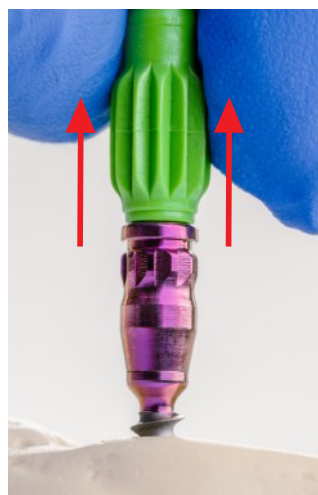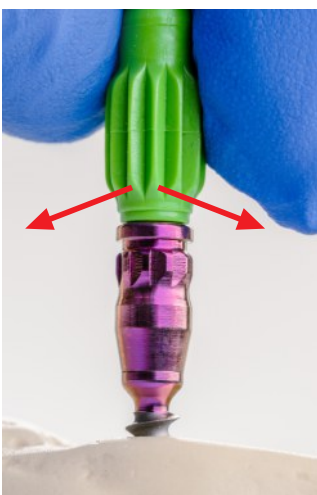

8 Insert implant via HFCM with implant driver HE **M P**.

Use implant driver ITHES/ITHE/
ITHEL/ITHEXL for ratchet TW50
with torque up to 50 Ncm

Or implant driver ITHEAO
for AO connection,
use ETAO handle




Implant insertion via HFCM carrier part – never exceed 93 Ncm

9 Unscrew SHFCM screw with screwdriver for implant driver HE **M P**
or with screwdriver **U** and take off HFCM.

Use screwdriver for implant driver
ITHES/ITHE/ITHEL/ITHEXL
and remove HFCM (with screw inside)

Or screwdriver
SD/SDL/SDXL
for ratchet TW50

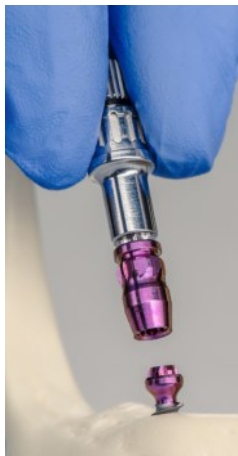


Or screwdriver SDH/SDHL/
SDHXL for handpiece,
use contra-angle handpiece



Or screwdriver SDAO
for AO connection,
use ETAO handle



Remove the screw and implant carrier

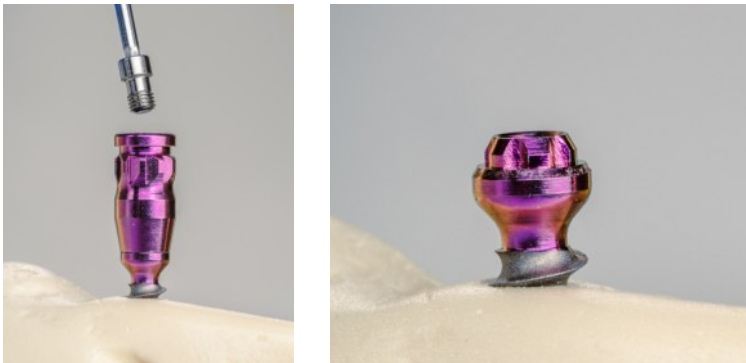

- ⑩ Insert implant via direct insertion with implant drivers **MP** .
Use implant driver ITM0/ITM10/ITM/ITML/ITMXL for ratchet TW50
with torque up to 50 Ncm



Implant insertion via direct insertion
never exceed 133 Ncm

11 Implant inserted in crestal position

