

ROOTT library integration in BlueSkyPlan software

Instructions that help you to import ROOTT library in BlueSkyPlan software



BlueSkyPlan.com


Powered by BlueSkyBio.com




ROOTT

Step 1. Download a new catalog using the link: <https://bit.ly/3h0XAzl>

Step 2. Open BlueSkyPlan software, click **Tools - Manage catalogs**

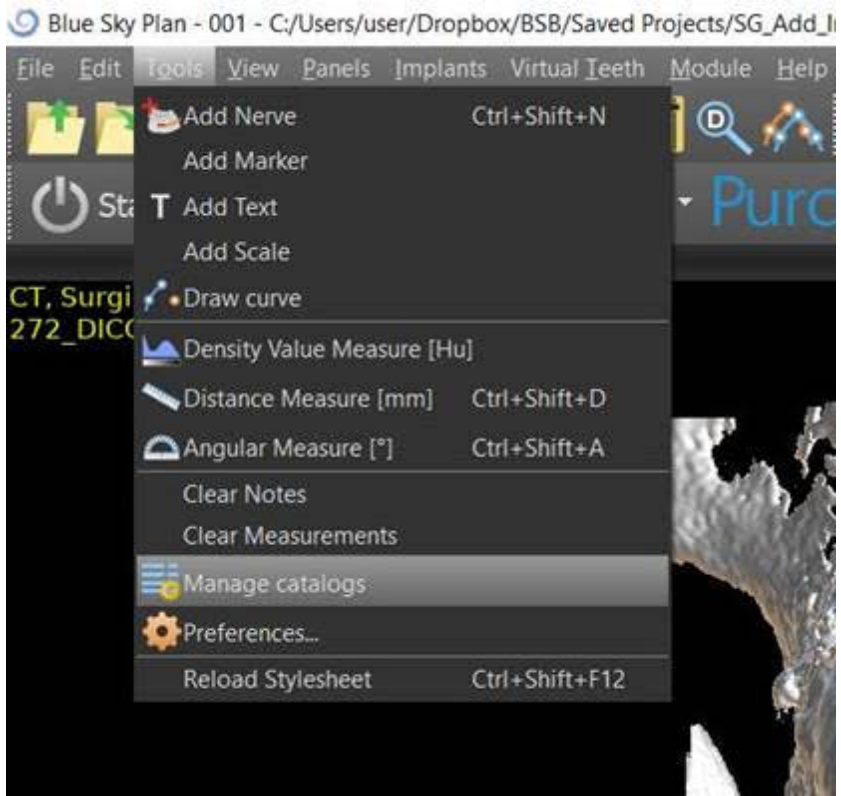

Step 3. Click **Import Catalog ...**

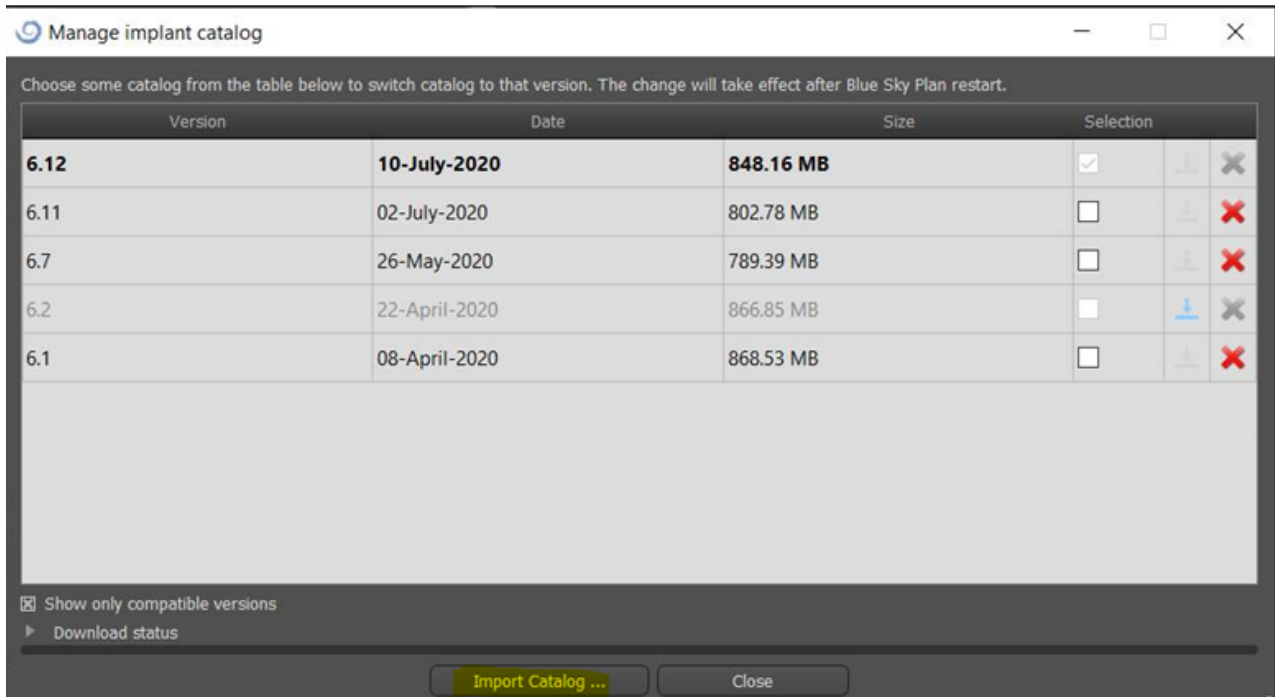


Step 4. Choose downloaded file and click check mark in **Section** column

Version	Date	Size	Selection		
7.8	11-March-2021	752.89 MB	<input checked="" type="checkbox"/>		
7.6	12-February-2021	730.03 MB	<input type="checkbox"/>		
7.5	22-January-2021	697.40 MB	<input type="checkbox"/>		
7.3	18-December-2020	1.06 GB	<input type="checkbox"/>		
7.0	29-October-2020	876.82 MB	<input type="checkbox"/>		


Step 5. Restart the system.

Optional. Use BlueSkyPlan project that has a DICOM and loaded lower Jaw Model for testing:
<https://bit.ly/3oGSNFR>


Step 6. Uncover hidden parts by clicking **Add New Implant**

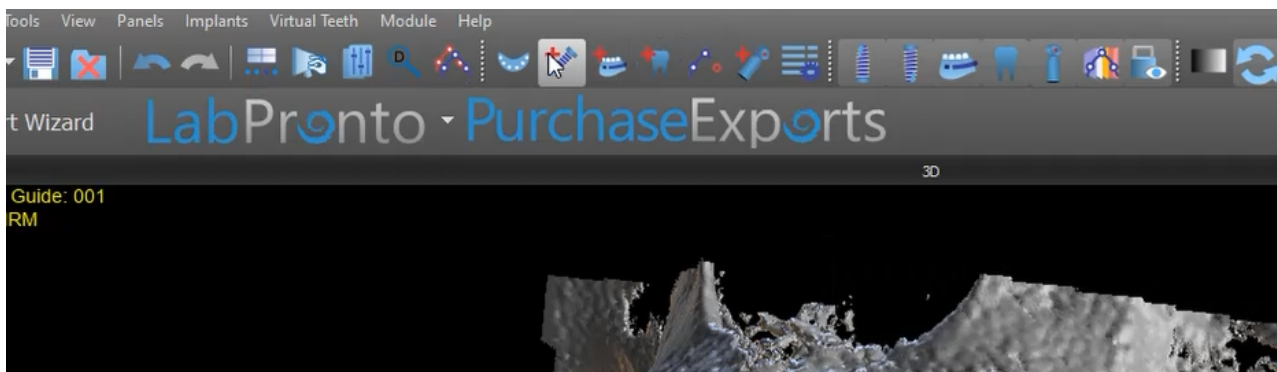

Step 7. Hold **Ctrl Shift** and click **Cancel** button with the mouse. A new window shall appear.

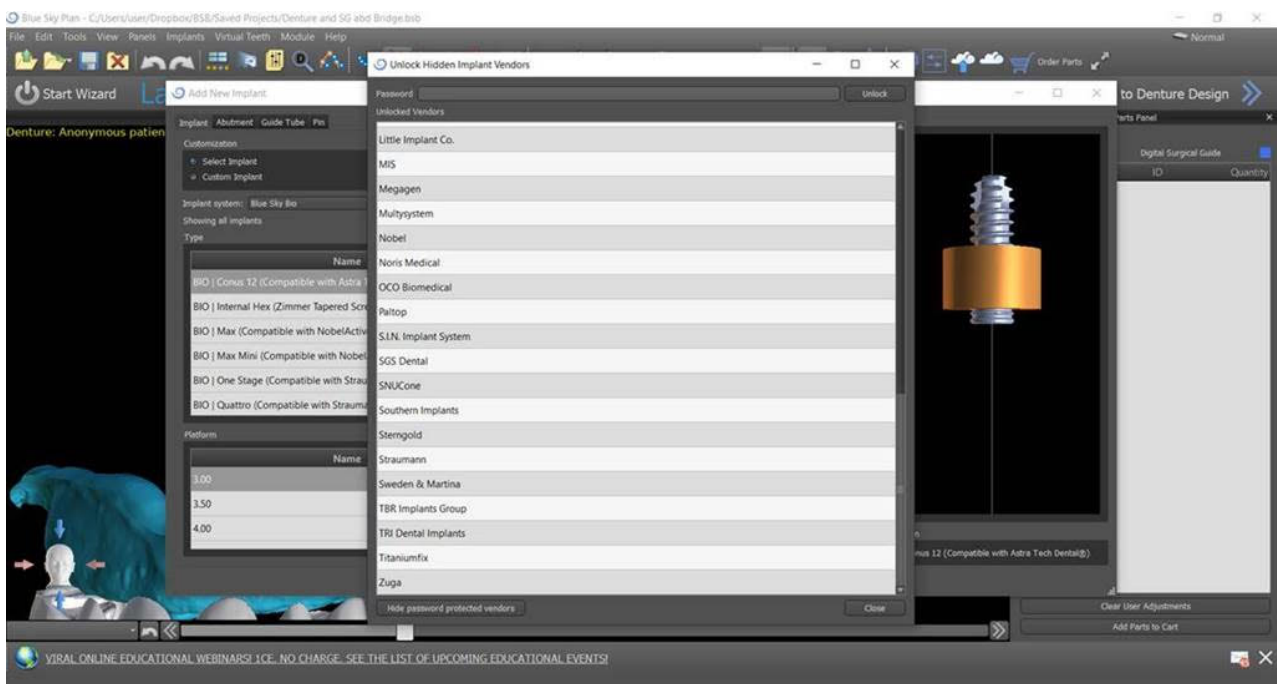

Step 7. Type **BSPTrate2021** inside Password window and click **Unlock**.

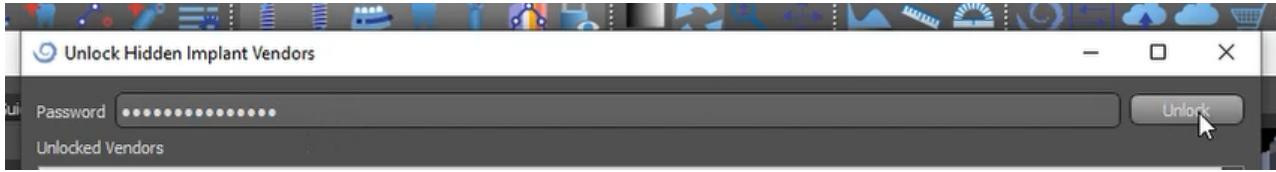

ROOTT library now is visible in
BlueSkyPlan software

For question please contact: msaltzman@blueskybio.com