

INSTRUCTION FOR OPENING BOXES AND BLISTERS OF ALL ROOTT NON STERILE PRODUCTS



Make sure the package is not damaged before use.
Do not use if package is damaged.

All ROOTT superstructures, instruments and drills are packed in blisters and boxes. Box consists of 1 blister with 5 separate parts, for easier management.



In each part can be:

a) 1 piece or b) 5 pieces



One part of the blister can be break out and opened separately.
Blisters can be removed one by one separately:



Instruments that are longer than 41 mm are packed in a plastic box:



Prepare a round bowl, instrument tray or soaking tube:



SUPERSTRUCTURES AND SMALL INSTRUMENTS

① Take a box



② Open box



③ Take blister out



④ Open blister

a) Open one part of a blister




b) Break out one blister and open



LONG INSTRUMENTS



① Take a box

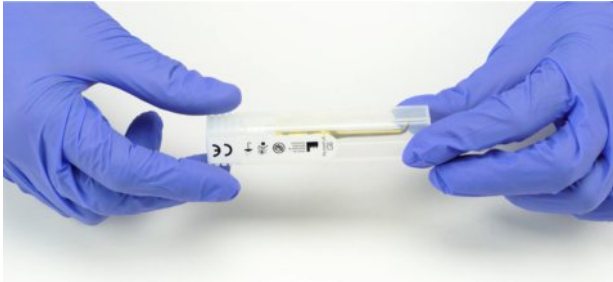

② Open box



③ Take instrument out

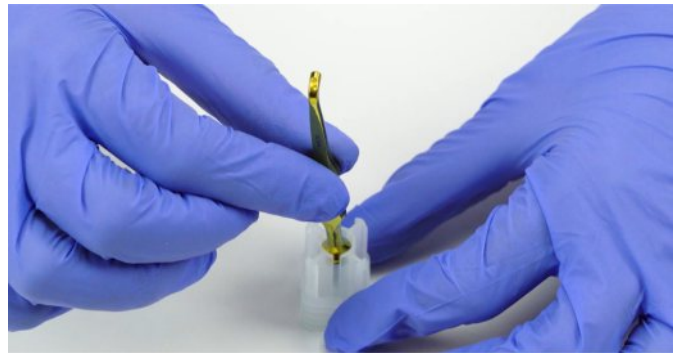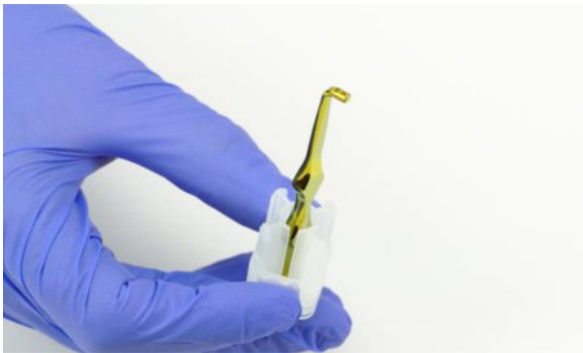

DRILLS



Risk of injury!
Be careful with sharp drills.
Use tweezers to reduce risk.



① Take a box



② Open box and take blister out



③ Open blister



Superstructures, drills and instruments should be washed,
disinfected and dried before sterilization.