

Examples of inspection of the defects

TRATE AG does not define the maximum number of uses appropriate for reusable instruments. The useful life of these devices depends on a number of factors including the methods and duration of each use and the handling between uses.

Products should not be used if are visible these defects:

- Corrosion, rusting;
- Pitting, discoloration;
- Cutting surfaces become blunt, are damaged, increased susceptibility to corrosion;
- Destruction of the material surface, removal of oxide layer increased susceptibility to corrosion;
- Damage of the instruments, especially of cutting surfaces increased susceptibility to corrosion.

Examples are shown for some instruments that can't be used anymore.

Screwdriver tip deformed



Implant driver discoloration



Implant driver corrosion



Drills corrosion and discoloration

