

## Biohazardous Implant-related Waste Disposal Instruction for the Dental Offices

This instruction applies to all Dental Implant System ROOTT products, produced by TRATE AG. Dental Implant System ROOTT is a system of endosseous dental implants with corresponding abutments, gingiva formers, covering and fixing screws, other prosthetic parts and associated surgical instruments (hereinafter - products).

This document describes the general waste management procedures for dental offices working with implant-related waste which is determined as biohazardous implant-related waste.

Biohazardous implant-related waste for TRATE AG means implant-related products which were in use and in direct contact with Human Body Fluids and Tissues.



Please note that the proper disposal of products protects the environment. It prevents possible harmful effects on humans and the environment, which could be caused by improper handling of waste.



Disposal of the biohazardous waste shall follow local regulations and environmental requirements, taking different contamination levels into account.



Do not dispose of biohazardous implant-related waste as unsorted municipal waste.

Waste disposed of in biohazard waste accumulation sites must be collected by service provider for final off-site treatment / incineration / disposal through a biowaste vendor in accordance with the local regulations and environmental requirements.

The waste holder who has transferred the waste to the biowaste vendor must have a document confirming the transfer of the waste (where the type of waste transferred, waste code and weight, date of waste transfer must be indicated).

<b>Examples</b>	Explanted implants*, gingiva formers (healing abutments), abutments, caps, transfers, screws, carrier
<b>Container</b>	Use a container provided by your biowaste vendor. Container must be rigid, leak resistant, have tight fitting lid, and labels on lid and all lateral sides with "Biohazard Waste" or biohazard symbol & "Biohazard" wording
<b>Storage</b>	Dispose of biohazard waste within the timeframe provided by biowaste vendor
<b>Packaging</b>	All products should be packaged in accordance with the infection control requirements under all national and legal requirements relevant Health and Safety legislation, legislation on waste.
<b>Labeling</b>	Affix biohazard symbol on lid & sides of secondary container.
<b>Treatment</b>	Off-site treatment
<b>Disposal</b>	Primary containers shall be properly closed. Transfer closed containers into large secondary containers in the waste accumulation site.

\*only related to the implants which are not a subject of the complaint

### Complaint-related products

If products are the subject of the complaint, do not dispose of this product and follow the requirements of Warranty and Return Policy and Instruction for product return

### Validity

Upon publication of these instructions for use, all previous versions are superseded.



## Biohazardous Implant-related Waste Disposal Instruction for the Dental Offices

### Please note

For the purpose of legibility, TRATE does not use ™ or ® in the text. This does not affect TRATE's rights with regards to registered trademarks.

### Change history:

Ver	Date	Change description	Approval
01	2020-10-23	Printing date	V. Shulezhko

### Approvals

Developer	Reviewer	Approval
Vladlena SHulezhko	Skaidre Janusaitiene	Siarhei Shulezhka
2020-10-23	2020-10-23	2020-10-23